


A505-2 MASSAGE CHAIR

SAVE THESE INSTRUCTIONS

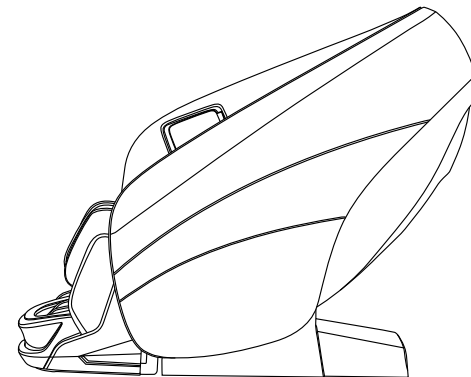

Thanks for purchasing this product. Please read this manual instruction carefully in order to know how to use this product correctly before operation. Please pay more attention to some important safety warning and keep this manual book properly for future use.

Note: Our company reserves the right to change and explain the design of the product. If there is some difference between the product function in market circulation and the description in the manual, it is a normal update of the product, and our company will not inform you otherwise. The specific product function and color shall prevail in kind.

CONTENTS

Before use

Safety precautions	02-05
Installation parts	06
Place massage chair	07-08
Before use	09
Product structure	10-13

Usage method

Operation instructions	14-21
------------------------------	-------

Maintain



Cleaning and maintenance	22
--------------------------------	----

Important


Troubleshooting	23
Product specifications	24

Safety Precautions (Must be observed)


- Please read all relevant instructions carefully before use.
- The notes marked here will help you to use the product safely and correctly.
- In order to clearly indicate the extent of harm and damage, precautions are divided into **"Warning"** and **"Caution"** that may be caused by incorrect use. Both of them are important for safety and must be followed.

 Warning	Indicates that serious injury or death may result from improper use.
 Caution	Indicates that misuse may cause minor injury or property damage.

- Please pay attention to the symbol before each safety item.


 Forbidden	Absolutely Prohibited
--	-----------------------

The following populations should not use this product


 Forbidden	<ul style="list-style-type: none"> ■ This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. ■ Children should be supervised to ensure that they do not play with the appliance. ■ The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance. ■ Patients with osteoporosis. ■ Patients with severe heart disease and wearing implantable medical devices such as pacemakers and defibrillators that are susceptible to electromagnetic waves. ■ People who have a fever (above 38 degrees). ■ Pregnant women or shortly after childbirth and during their menstruation period. ■ People who follow medical advice and are prohibited from receiving massage due to thrombosis, aneurysm, severe varicose veins, or other circulatory diseases. Within two weeks after the appearance of blood clots, it is strictly prohibited to massage any form of body parts that have blood clots. ■ People with injuries or skin conditions. ■ Unsupervised children under the age of 14, or anyone unconscious ■ People advised by their doctors to rest or those feeling unwell. ■ People with spinal abnormalities or scoliosis are prohibited from receiving this product due to medical advice due to other diseases. ■ People with wet bodies.
---	---


Safety Precautions (Must be observed)

Important safety warnings


 Warnings	<ul style="list-style-type: none"> ■ If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard. ■ Keep children away from the movable parts of this product. ■ Use a properly grounded power source that is compatible with this device. ■ Unplug the power cord after use or before cleaning to prevent personal injury or damage to the device. ■ Use this device according to the instructions in the manual. ■ Do not use any parts or accessories that are not recommended. ■ Do not use this product outdoors. ■ Do not use near open flames such as stoves, and do not smoke while using. ■ Do not use heating equipment such as electric blankets on this product. ■ Do not sit on the back of the chair, leg massagers, shoulders, armrests, or stand on the seat. ■ Please read the instruction manual before operation. ■ Never use this product for any purpose not listed in the manual. ■ Limit each use to 20 minutes. ■ Do not use this product if the cover or leather is broken or damaged. ■ Do not drop anything into this product. ■ Never fall asleep while using this product. ■ Do not use this product if you are intoxicated or feeling unwell. ■ Do not use this product within one hour of eating. ■ Do not apply excessive force when using this product to avoid injury. ■ Do not use for purposes other than massage. ■ Precautions: Keep away from children. Do not store near high temperatures or open flames. ■ Fully insert the power plug into the outlet and ensure secure contact during use. ■ When using an extension cord, only use nationally certified products. ■ Disconnect power when leaving the premises. ■ Immediately discontinue use and disconnect the power if abnormal noise, odor, overheating, or malfunction occurs. ■ Do not power the unit from lamp socket outlets. ■ Total current draw of connected appliances must not exceed the rated current of the main outlet/circuit. ■ Never use near flammable/explosive materials. ■ Do not disassemble this product or modify its internal wiring, components, or safety features. ■ When unplugging, do not pull on the power cord. ■ Only use specified fuse ratings - never substitute with conductive materials or incorrect ratings. ■ Do not alter the trip settings of circuit breakers or residual current devices.
--	---


Safety Precautions (Must be observed)


Safety precautions	
 Caution	<ul style="list-style-type: none"> ■ Check whether voltage is consistent with the specification of this product. ■ Never insert or unplug the power cord with wet hands. ■ Do not allow water to enter this product to avoid electric shock or damage. ■ Do not pull the power cord when plugging or unplugging. Avoid rough handling. ■ Do not damage the wire or modify the circuit of this product. ■ Do not clean electrical parts, such as the power switch or plug, with a wet cloth. ■ Stay away from this product during a power failure to avoid injury if power is suddenly restored. ■ Stop using this product immediately if it malfunctions, and contact your local supplier. ■ Stop using this product if you feel unwell, and consult your health care practitioner.

Product care and maintenance	
 Caution	<ul style="list-style-type: none"> ■ This product should be maintained by the service center designated by the manufacturer only. No user should disassemble or maintain it without permission. ■ Do not forget to turn off the main power switch after usage. ■ Do not use this product if the power socket is loose. ■ If the product will not be used for an extended period, roll up the power cord and store it in a dry, dust-free environment. ■ Do not store this product at a high temperature or near open flame. Avoid prolonged exposure to direct sunshine. ■ Please clean this product with dry cloth. Never use thinner, benzene or alcohol. ■ The mechanism of this appliance is specially designed and does not require special maintenance. ■ Do not puncture this product with sharp objects. ■ Do not roll or pull this product on any uneven ground. Instead, it should be lifted before moving. ■ Please use this product intermittently. Do not keep it working continuously for a long period of time. ■ Allow intermittent operation - continuous long-term use is prohibited.

Safety Precautions (Must be observed)

Storage conditions	
 Caution	<ul style="list-style-type: none"> ■ Storage room temperature: 5 °C to 40 °C. ■ Relative humidity (RH) between 20% and 80%; free of corrosive gases, and a well-ventilated room.



Operating environment	
 Caution	<ul style="list-style-type: none"> ■ Do not use this product in a highly humid environment, such as a bathroom. ■ Stop using this product immediately when the ambient temperature changes sharply ■ Do not use this product immediately after a sudden change in ambient temperature. ■ Do not use this product in environments with heavy dust or corrosive gases. ■ Do not use this product in confined spaces or areas with poor ventilation.

Solutions to ordinary malfunctions	
 Caution	<ul style="list-style-type: none"> ■ If the motor sounds during operation of this product, this is the normal operation sound. ■ If the controller is not working normally, check if the power plug and socket are securely connected, and ensure the power switch is turned on. ■ If the rated working time is over, this product will automatically activate the power-off button; If this product keeps operating for a long period of time, the temperature protector will stop this product automatically. The product can be used again after half an hour. ■ Weight limit: 120kg. Heavier users should purchase our reinforced models.


Installation parts

Calf mechanism installation instructions

1. Align and securely insert the calf plug and air tube into their corresponding ports located at the front end of the steel frame assembly (Figure 1).
2. Once all connections are complete, open both calf retention covers on the sides of the steel frame (Figure 2).


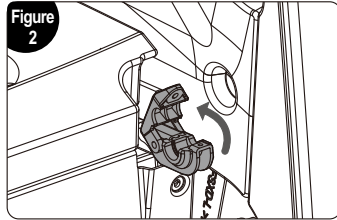
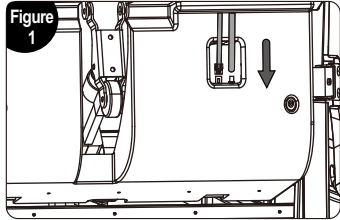


3. Raise the calf mechanism and put it into the slot and cover the fixing cover and finally fix it in place with 2 M4 screws (Figure 3)

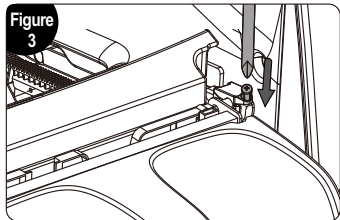

Fragrances Installation Instruction

1. Rotate the fragrance cover clockwise until it detaches from the magnetic locking mechanism (Figure 1).
2. Remove the cover to access and replace the fragrance disc (Figure 2).
3. To reinstall, align the cover, press down, and rotate it counterclockwise until it locks securely and sits firmly in place without wobbling (Figure 3).

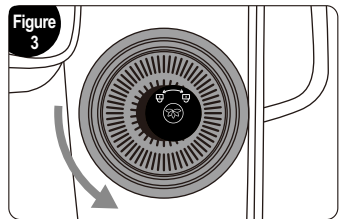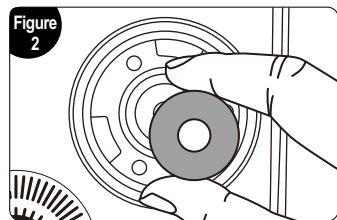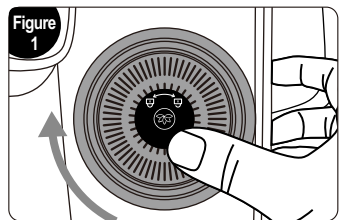


Place massage chair

Installation placement

Make sure there is enough space for the massage chair can be tilted at least 5 cm.



Do not expose the massage chair to direct sunlight or high-temperature environments (e.g., in front of heating devices). Place it in an area where it can avoid heat and direct sunlight. For long-term use in a fixed location, it is recommended to place a soft cushion underneath. The cushion should be large enough to cover the area where the chair and the footrest touch the floor.

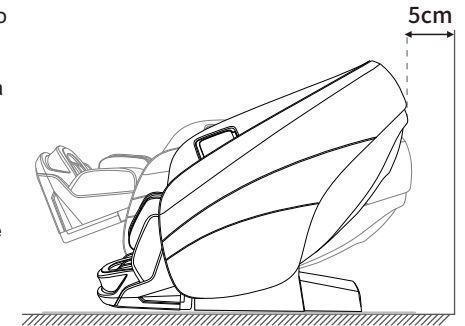


Product lift instruction

Two-person lifting required due to product weight.



1. Please lift the massage chair when moving it on delicate surfaces, such as wooden floors.
2. When moving on similar surfaces, lifting is recommended to avoid damaging the casters. Be cautious when lowering the chair into the designated position to avoid hand or foot injuries. Do not release the chair until it is completely flat. To prevent floor damage, it is recommended to place a mat or protective covering on the floor.

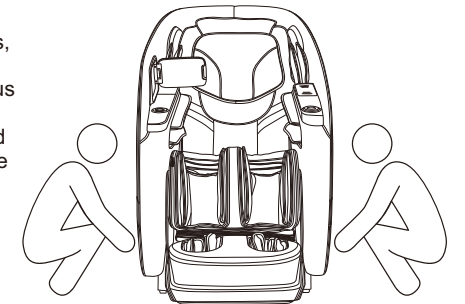

Place massage chair


Move method

Use the rear casters to move the massage chair. Tilt the chair backward approximately 45° as shown in the diagram to move it.

NOTE: Excessive force to title will cause the chair to tip over completely and product damage.





1. Before moving, ensure the massage chair is properly powered off, with both the backrest and calf sections fully reset, and the power supply disconnected.
2. The chair should remain empty during movement; no people, pets, or objects should be on it.
3. Do not move the chair over obstacles higher than 2mm or gaps wider than 5mm.
4. When moving the chair using the casters, avoid continuous movement over distances greater than 50 meters. If necessary, move the chair slowly and at a steady pace.

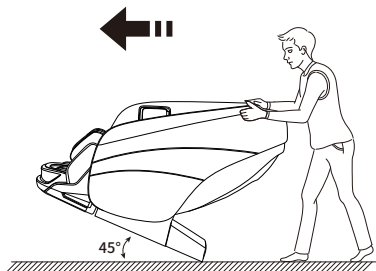

How to disassemble and use headrest cushion and backrest cushion

Using a headrest cushion can reduce the intensity of kneading massage on the neck and shoulders, and you can decide whether to use a headrest cushion according to your needs (recommended to use a headrest cushion). The backrest cushion and the backrest are connected by a zipper (1), and the headrest cushion and the backrest cushion are connected by a zipper (2).

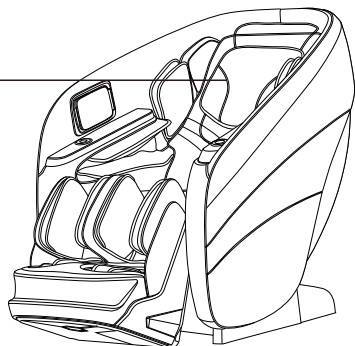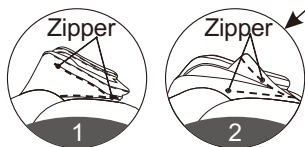


Before use

Check around




Make sure that there are no people, pets and other objects around the unit.

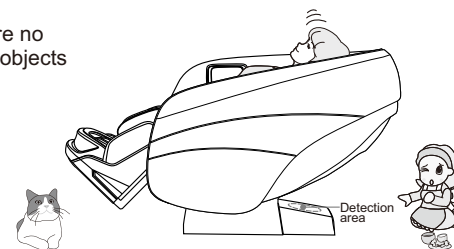




Make sure there are no foreign objects in the gaps of the machine



Before massaging, make sure that no foreign objects are caught in the legs, feet, etc. Under no circumstances allow hands, feet, and head to penetrate into the area between the lower leg mechanism and main body, or the area between the armrest and the capsule. Under no circumstances should hands, feet, or head sink under the fabric or leather covering of the massage area.

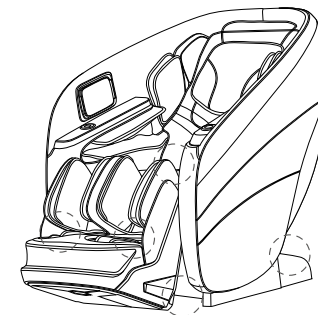


Turn on the power

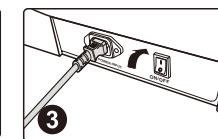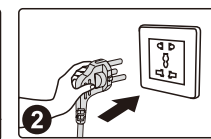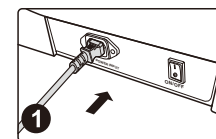

- Please check the power cord and plug for damage before use.
- Please supply power to this product in accordance with the working power requirements of this product's calibration.
- It is strictly forbidden to use a power supply other than the nominal power supply of this product.
- It is strictly forbidden to connect various power conversion devices without authorization to supply power to this product.

Product structure


Description of Components

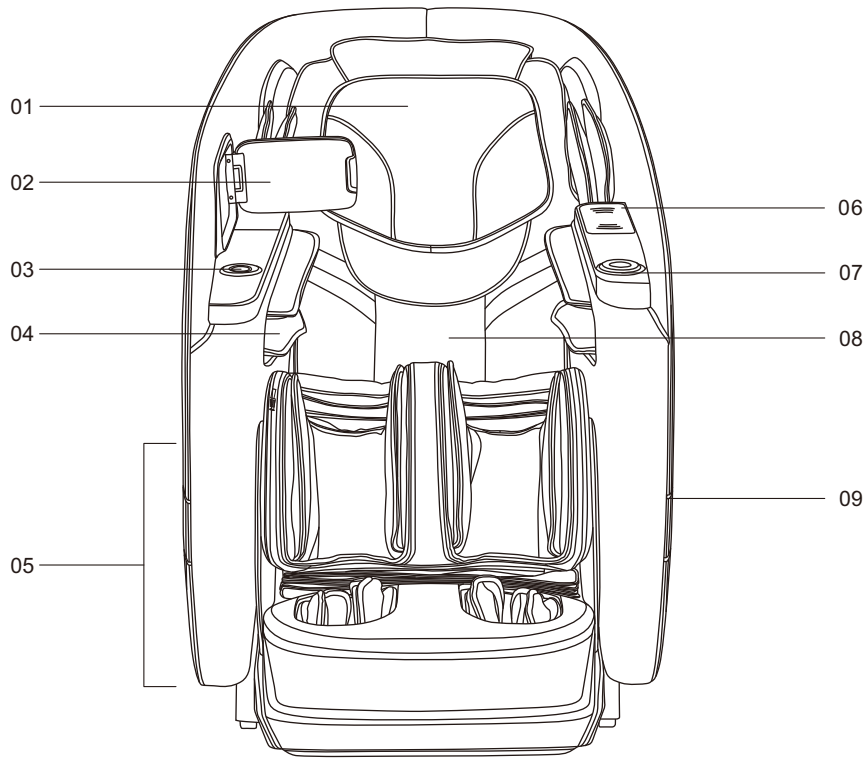

- | | |
|-------------------------|-----------------------------|
| 01.Headrest | 06.USB A+C |
| 02.Control Panel | • Dual Charging |
| 03.Fragrances | 07.Armrest Shortcut Keys |
| 04.Armrest | 08.Backrest Cushion |
| • Air Pressure Assembly | 09.Armrest Ambient Lighting |
| 05.Leg | |
| • Massage Mechanism | |

Product structure


Schematic view of functional distribution

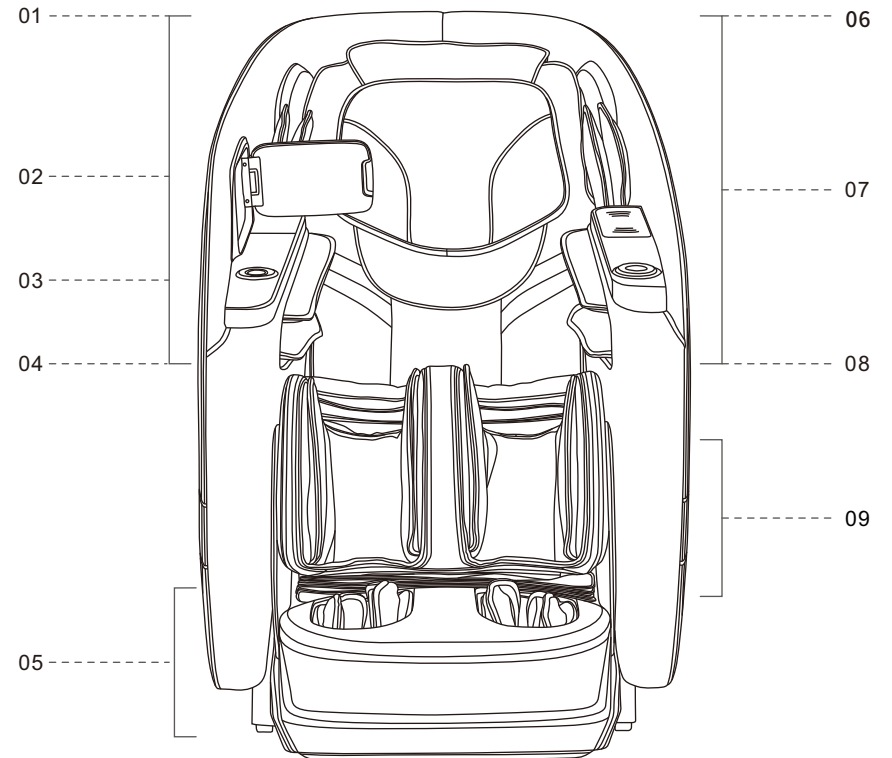

- | | |
|-------------------------------------|---------------------------|
| 01.Both Sides of Headrest | 06.Both Sides of Shoulder |
| • Voice Control | • 3D Digital Audio |
| • Built-in Voice Control Microphone | • Music Playback |
| • Voice Announcement | • Bluetooth |
| 02.Shoulder Areas (Both Sides) | 07.Back Area |
| • Airbag Massage | • Heating |
| 03.Head/Neck to Seat Area | 08.Left Arm Area |
| • Core Massage | • Airbag Massage |
| 04.Right Arm Area | 09.Leg Area |
| • Airbag Massage | • Airbag Massage |
| 05.Foot Area | |
| • Airbag Massage | |
| • Foot Roller Massage | |
| • Anti-Pinch Sensor | |
| • Ankle Airbag Node Massage | |
| • Motorized Footrest Extension | |
| • Automatic Leg Length Detection | |


Product Structure

Controller Key Instructions

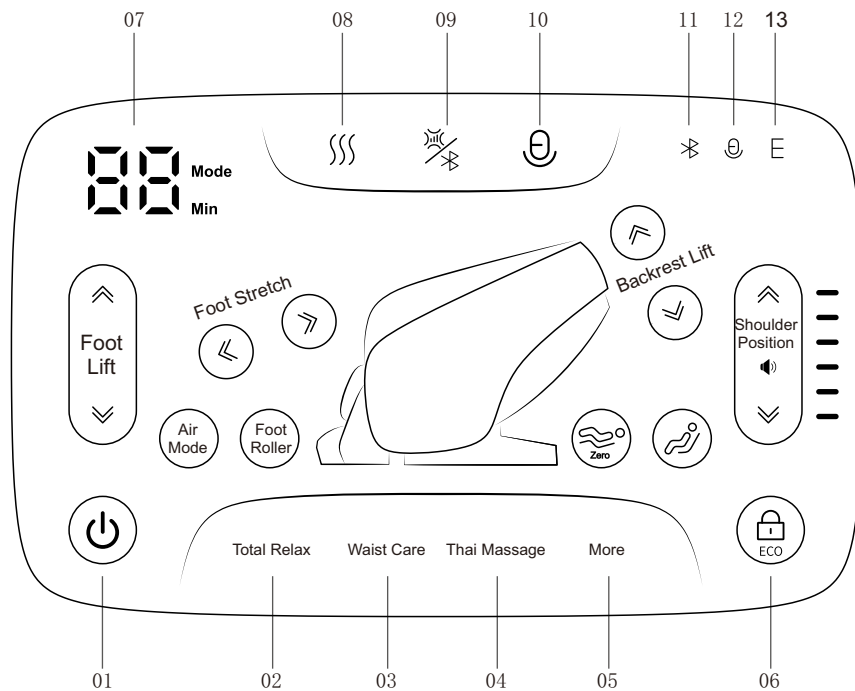

01. Power Key: Press this key to turn on the massage chair. During operation, briefly press to pause. When powered on, press and hold for 3 seconds to turn off the massage chair.
02. Zero Gravity Key: Activates zero gravity function.
03. Waist Care Key: Activates waist care program.
04. Thai Stretch Key: Activates Thai stretching program.
05. Program Select Key: Press repeatedly to select different massage program.
06. Lock Screen Key: Short press to turn the screen lock function on/off. When locked, the manual controller keys will be disabled. Long press to turn the low-power switch on/off.
07. Digital Display: Shows massage countdown timer and automatic program mode.
08. Heat Key: Turns heating function on/off.
09. Air Pressure/Bluetooth Key: Short press to adjust air pressure intensity through 1-5 levels. Long press to turn the Bluetooth function on/off.
10. Voice Control Key: Short press to wake up the voice control assistant. Long press to turn the voice control function on/off.
11. Display Bluetooth on/off status.
12. Display voice control on/off status.
13. Displays low-power mode on/off status.


Product structure

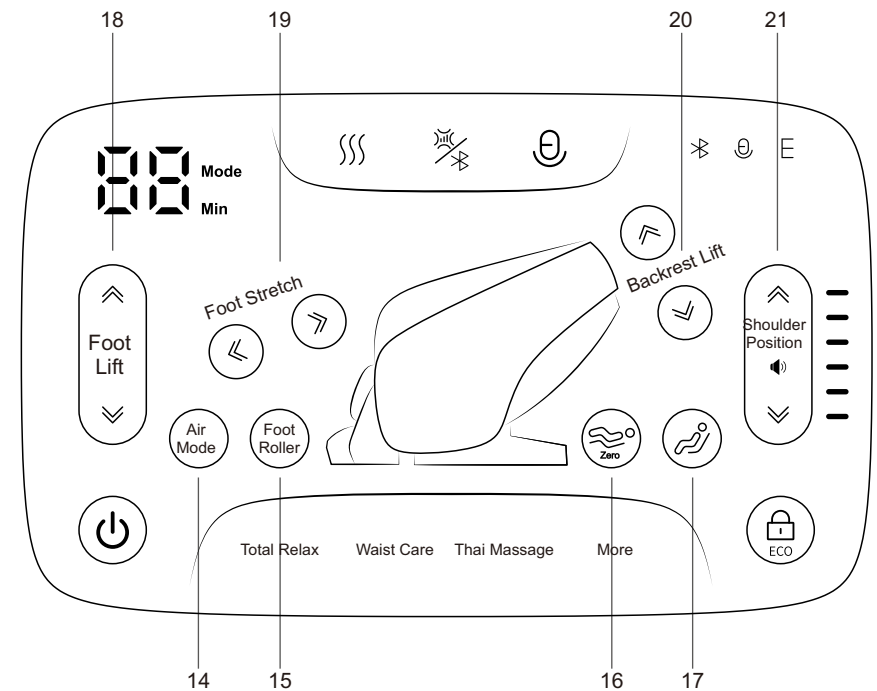

14. Air Pressure Mode Key: Press this button to turn the air pressure massage function on/off, cycling through upper body air pressure, lower body air pressure, full body air pressure, and turning off all air pressure functions.
15. Foot Roller Key: Turns the foot roller massage function on/off, with three adjustable speed levels (Low/Medium/High) that cycle sequentially between settings and off.
16. Zero Gravity Key: Adjusts zero gravity angle settings, cycling through Zero Gravity I, Zero Gravity II, Zero Gravity III, and Off.
17. Reset Key: Determines whether the massage chair resets to default position after the timer ends. When activated, the massage chair will return to its default angle when the timer completes.
18. Footrest Adjustment Key: Press the up/down arrows to raise or lower the footrest.
19. Footrest Extension Key: Press the up/down arrows to adjust the footrest telescoping function (shorten/extend).
20. Backrest Adjustment Key: Press the up/down arrows to adjust the backrest angle.
21. Shoulder Position/Volume Key: Briefly press the up arrow to adjust the shoulder massage position upward, press and hold for 3 seconds to increase the volume; briefly press the down arrow to adjust the shoulder massage position downward, press and hold for 3 seconds to decrease the volume.

Operation Instructions

Armrest Shortcut Keys

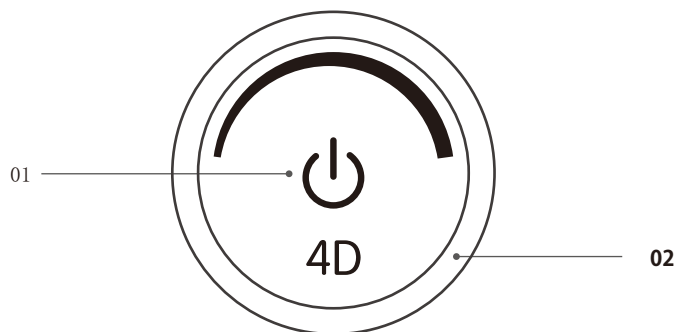

01. Power Button: Press and hold for 3 seconds to turn the massage chair on/off. During operation, briefly press to pause/resume.
 02. 4D Adjustment Knob: Rotate clockwise to increase 4D intensity (maximum level 5). Rotate counterclockwise to decrease 4D intensity (minimum level 1).

I. Start Massage

1. Press the power key on the hand control panel or press and hold the power button on the left armrest to turn on the massage chair.
2. Automatic Angle Adjustment: After selecting an automatic program, the massage chair will automatically adjust to the zero-gravity position. Wait for the adjustment to complete before proceeding.
3. Wait for the body detection to complete, or select automatic mode to start the massage directly.
4. After body detection, a voice prompt will guide you to fine-tune the shoulder position. If the massage rollers are already at your shoulder position, no adjustment is needed. If not, briefly press the up or down key to adjust the rollers to your shoulder position, then start the massage. If no action is taken within 10 seconds, the current roller position will be set as the default shoulder position, and the massage will begin.
5. Pause Function: After powering on, briefly press the power key to pause all massage functions. Press the power key again to resume the previous massage state. Note: In pause mode, only the power-off function is available; all other functions are disabled.

II. Automatic Program (Please use this function after powering on)

1. Automatic Program Selection
 - 1.1 Press the Automatic Program key on the manual controller to select and start a massage program.
 - 1.2 There are 20 auto programs in total, 3 featured programs have dedicated shortcut buttons for direct access, the remaining programs can be cycled through using the "More Programs" key.
- Featured Programs: Total Relax, Waist Care, Thai Massage.
 Other Programs: Shoulder massage, Back Massage, Waist Massage, Hips Massage, Neck & shoulder Care, Chinese Massage, Waist & buttock Shape, Leg & foot Relieve, Superior Enjoyable, President Cosy, Keep Fit, Office Regimen, Energy Boost, Lunch Break, Ache relieve , Sweet Dream, Relax massage

Operation Instructions

Function	Description	Display
Total Relax	With the aid of flexible guide rails, let the body posture angle closer to lying flat; with the effective anti-arch stretching of 4D movement to the back, achieve zero pressure stretching, effectively relieve lumbar muscle pain, and eliminate fatigue, thus to get a better massage experience, relieve the stress on the body and mind.	01
Waist Care	Massage the waist by means of kneading, patting and medical massage mainly, while the movement holds against the back and waist for 4D medical massage, so as to relieve lumbar spine pain; combined with the thermostatic heat configuration around the waist, maintain a comfortable temperature to bring warm massage experience; with gentle and moderate massage intensity and comfortable temperature, bring warm massage experience.	02
Thai Massage	Mainly use calf and backrest electric push rod to clamp legs, hands and shoulders with the aid of air massage, and then apply stretching massage, while the movement holds against the back and waist for 3D medical massage, so that the body is in the Thai stretching state. It is suitable for people under high stress to improve their head-down on chest and humpback. Massage intensity is strong.	03
Shoulder massage	Special massage techniques for neck and shoulders target the cervical vertebrae and shoulder blades to relieve neck and shoulder soreness and reduce fatigue.	04
Back Massage	Focuses on back massage to relieve back soreness.	05
Waist Massage	Focuses on waist massage to relieve lower back pain.	06
Hips Massage	Mainly uses kneading and Chinese massage techniques to massage the hips, followed by tapping and kneading techniques to relax hip muscles and provide a lifting effect. Suitable for women's massage, it also has maintenance benefits for male prostate.	07
Neck & Shoulder Care	Massage the neck and shoulders by means of kneading and stretching techniques mainly, and also the upper back by medical massage, and then press the Jianjing acupoint to relieve cervical spondylosis and achieve the effect of relaxing the muscles of the neck and shoulders. Massage intensity is gentle, suitable for people working in office and suffering from cervical fatigue.	08

Operation Instructions

Chinese Massage	By means of Traditional Chinese medicine massage technique mainly, apply 4D massage on the back and waist so as to promote the blood circulation of the back and waist, soothe the meridians and relieve the blockage of the back and waist through massage. It is suitable for people with back and waist pain, and lumbar muscle strain. Massage intensity is gentle.	09
Waist & Buttock Shape	Massage buttocks by kneading and medical massage mainly, and then through patting and kneading, deeply massage buttocks and perineum, to relax the hip muscles and lift the hip, which is suitable for women massage, and also for men for maintenance of prostate.	10
Leg & foot Relieve	Massage the shoulders and waist by means of kneading, patting, kneading and patting mainly, apply air massage on the legs and feet for squeezing massage, allowing more soothing leg and foot massage, suitable for sedentary people with low stress on the legs and feet, not only relieving the cervical spine, lumbar spine pain of sedentary people, but also soothing the legs and feet. Massage intensity is gentle.	11
Superior Enjoyable	Apply air massage on the whole body, with the foot roller and calf kneading massage. Calf stretching function is canceled, suitable for the elderly and people suffering stress on back. The massage intensity is gentle.	12
President Cosy	Massage the shoulders, neck, back and waist by means of kneading, patting, kneading and patting mainly, to relieve fatigue, promote blood circulation, and release stress by such deep massage. It is suitable for men, and the massage intensity is moderate.	13
Keep Fit	Massage the shoulders, neck, waist and hips by means of kneading and finger pressing techniques, through the massage on the whole body, to relax the body and mind, and enjoy the queen-level treatment. It is suitable for women, and the massage intensity is gentle.	14
Office Regimen	Massage the shoulders, neck and waist by means of kneading, patting, kneading and patting techniques mainly, to relieve shoulder and neck pain, improve lumbar muscle soreness and maintain lumbar spine through such deep massage. It is suitable for office white-collar and sedentary people, and the massage intensity is moderate.	15
Energy Boost	Massage the waist, shoulders and neck by means of kneading, patting, kneading and patting techniques, to awaken the body's vital energy, stimulate physical energy through such deep massage of the whole body with the massage intensity being heavy, which is suitable for people with strong stress, such as athletes, for stimulating the potential before a race.	16


Operation Instructions

Lunch Break	Massage the back and waist by means of kneading and patting techniques, and press the Shenyu acupoint, to improve the quality of lunch break, so that the user is more energetic in afternoon work. It is suitable for office workers, and the massage intensity is gentle.	17
Ache Relieve	The massage primarily uses kneading, tapping, and shiatsu techniques to target the neck, shoulders, back, and lower back. It also incorporates air pressure massage for the calves, providing relief to the knee joints. Deep massage helps alleviate soreness and discomfort, making it ideal for individuals with back and waist pain. The intensity is moderate and comfortable.	18
Sweet Dream	Massage the shoulders, neck, back and waist by means of kneading and medical massage techniques, and press the acupoints of Fengchi and Shenyu, to improve the sleep quality and help sleep. It is suitable for people suffering from poor sleep, and the massage intensity is gentle.	19
Relax Massage	The massage primarily uses kneading and stretching techniques for the neck and shoulders, combined with tuina for the upper back. By applying pressure to the Jianjing acupoint, it helps relieve cervical spondylosis and relax the neck and shoulder muscles. The gentle intensity makes it suitable for office workers and those experiencing neck fatigue.	20


III.Angle Adjustment (Please use this function after powering on)

During operation, you can adjust the massage angles as needed, including zero gravity, footrest extension, backrest elevation, footrest lift, and reset functions for optimal positioning.

1.Zero Gravity Adjustment


Key	Description
	Three adjustable levels, corresponding to three preset massage chair angles. Click the Zero Gravity icon to cycle through: Zero Gravity 1, Zero Gravity 2, Zero Gravity 3, and Off. Upon completing the zero gravity adjustment, the leg mechanism will automatically detect the calf length.

2.Footrest Extension Adjustment



Key	Description
	Footrest Retraction Key: Manually adjusts the footrest mechanism to retract. Press and hold to lift the footrest gradually; release to stop. Upon completion, the leg mechanism will automatically detect the calf length.
	Footrest Extension Key: Manually adjusts the footrest mechanism to extend. Press and hold to lower the footrest gradually; release to stop. Upon completion, the leg mechanism will automatically detect the calf length.

Operation Instructions


3. Footrest Lift Adjustment

Key	Description
	Footrest Lift Key: Press and hold to gradually raise the footrest mechanism; release to stop. Upon completion, the footrest mechanism will automatically detect calf length.
	Footrest Descent Key: Press and hold to gradually lower the footrest mechanism; release to stop. Upon completion, the footrest mechanism will automatically detect calf length.

4. Backrest Lift Adjustment

Key	Description
 	Backrest Lift Key: Press and hold to gradually elevate the backrest while simultaneously lowering the leg mechanism; release to stop both movements. Upon completion, the leg mechanism will automatically detect calf length.
	Backrest Descent Key: Press and hold to gradually lower the backrest while simultaneously raising the leg mechanism; release to stop both movements. Upon completion, the leg mechanism will automatically detect calf length.


5. Reset Adjustment

Key	Description
	Reset Key: Choose whether the massage chair resets after the timer ends. When turned on, the button backlight stays on and the chair resets to its default angle after timing. When turned off, the button backlight turns off and the chair maintains its current position after timing.

IV. Auxiliary Functions (Please use this function after powering on)


During operation, select auxiliary massage functions as needed, including foot rollers, air pressure modes, back heating, air pressure intensity, Bluetooth connectivity, and screen lock.

1. Foot Roller


Key	Description
	Foot Roller Key: Press this key to cycle through intensity levels 0 to 3 for the foot rollers. Level 0 turns off the foot rollers, while levels 1 to 3 activate them.

Operation Instructions


2. Air Pressure Mode

Key	Description
	Air Pressure Mode Key: Press this key to activate the air pressure massage and cycle through modes: Full-Body Air Pressure, Lower-Body Air Pressure, Upper-Body Air Pressure, and Off.


3. Back Heating

Key	Description
	Back heating key: Press this key to turn the back heating function on/off.


4. Air Pressure Intensity

Key	Description
	Air Pressure Key: Short press the "Air Pressure/Bluetooth" key to adjust air pressure intensity, cycling through levels 1 to 5.

5. Bluetooth

Key	Description
	<p>Press the "Air Pressure/Bluetooth" key to turn Bluetooth on/off. Bluetooth-enabled audio devices (such as smartphones or MID tablets) can pair with the massage chair's Bluetooth module to wirelessly stream music to the chair's audio system.</p> <ol style="list-style-type: none"> Access your smart device's main menu, open "Settings" → select "Bluetooth" → turn on "Bluetooth" → it will automatically search for new Bluetooth devices. The smart device will detect the corresponding Bluetooth model (e.g., IMCM-XXXX). Click the pairing key and select "Pair" when prompted to complete pairing. Open your smart device's music player and select songs to play.

6. Lock Screen

key	description
	Lock Screen Key: Press this key to turn the lock screen function on/off. When locked, the manual controller keys will be disabled. Long press the lock screen key to turn the low-power mode on/off. When low-power mode is on, the controller cannot power on the chair in standby mode—you must long press the armrest power key to turn it on.

Operation Instructions

V. Voice Control (Please use this function after powering on)

Press and hold voice control key to turn voice control on/off. When activated, the massage chair can be controlled via voice commands.

- 3.1 With voice control enabled and the chair powered on, wake the voice assistant by either saying "Hi Alice" near the right speaker or briefly pressing the Voice Control key. The assistant responds "I'm here" when successfully awakened, enabling voice control.
- 3.2 After activation, speak commands within 6 seconds. The assistant remains active for follow-up commands if spoken within 6-second intervals; otherwise, reawakening is required.

Voice Control Entry		
Entry: "Hi Alice" or "Hey Alice". Answer: I am here.		
No.	Voice control entry	Answer entry
01	Massage on	Answer: Ok,Massage on
02	Massage close	Answer: Ok,Massage close
03	Total Relax	Answer: Ok,Total Relax
04	Waist Care	Answer: Ok,Waist Care
05	Neck & shoulder Care	Answer: Ok,Neck & shoulder Care
06	Energy Boost	Answer: Ok,Energy Boost
07	Lunch Break	Answer: Ok,Lunch Break
08	Thai Massage	Answer: Ok,Thai Massage
09	Sweet Dream	Answer: Ok,Sweet Dream
10	Relax massage	Answer: Ok,Relax massage
11	Open the air pressure	Answer: Ok,Open the air pressure
12	Close the air pressure	Answer: Ok,Close the air pressure
13	Up the seat position	Answer: Ok,Up the seat position
14	Down the seat position	Answer: Ok,Down the seat position


Operation Instructions

VI. End Massage


During the massage:

1. Press and hold the power key on the manual control panel, all massage functions will turn off, and the chair will return to its default position.
2. Press and hold the power key on the armrest, all massage functions will turn off, and the chair will return to its default position.
3. Use the voice command to shut down, all massage functions will turn off, and the chair will return to its default position.
4. Timer completion, all massage functions will turn off. If the chair reset function is enabled, the chair will return to its default position; otherwise, the chair angle will remain unchanged.


Cut off the whole machine power supply, as shown in the figure (Schematic diagram of power switch position of the whole machine).

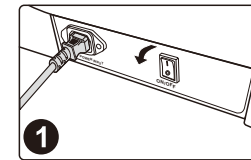

Turn off the power switch ("0" position)

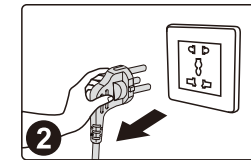

Schematic diagram of pulling out the power plug

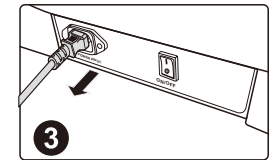


Remove the product connection power cord

Cleaning and Maintenance

Cleaning synthetic leather and plastic parts

Wipe the leather with a soft, dry cloth.

- Do not use chemicals such as thinner, gasoline, alcohol, etc. to clean the unit.
- When using commercially available leather care products (rags), please follow the instructions for use.
- If synthetic leather is particularly dirty, wipe it in the following way.
 - ① Immerse a soft cloth in water or hot water containing 3% to 5% neutral detergent, then wring it out thoroughly.
 - ② Use the above soft cloth to wipe the leather surface.
 - ③ Wipe off the cleaning solution with a cloth soaked in clean water and thoroughly wrung out.
 - ④ Wipe with a wrung soft cloth.
 - ⑤ Let it air dry naturally.
- When it is difficult to remove stains, please immerse commercially available melamine foam in neutral detergent, and then wipe the machine with it.
- Do not use a hair dryer to dry the surface.
- Be careful when wiping the machine with denim and colored fabrics, as the color of the fabric will stain the surface of the synthetic leather.
- Do not keep these parts in contact with plastic for a long time, as this will cause discoloration.
- Synthetic leather may be discolored, so when using hair dye, please cover the touched part with a towel.

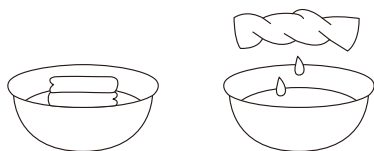

Troubleshooting

Fault	When the calf mechanism or the backrest reaches a certain position, press the corresponding adjustment button and it will no longer change, and a continuous "beep, beep" sound will be emitted.
Solution	Due to product structure design and ergonomic requirements, this product has a limited protection design for the limit state of attitude adjustment. At the same time, a reminder sound will sound when the limit is reached.

Fault	The function does not work or some functions do not work after the machine is turned on.
Solution	After the machine is turned on by pressing the switch key, it is necessary to manually select an automatic massage program or other working state. If it is not operated, the machine will automatically shut down after 20 minutes. If the manual operation mode is selected, the functions that are not selected may be disabled, and the working mode needs to be manually set in the manual mode.

Fault	The machine makes a leather rubbing sound when adjusting its posture.
Solution	This product adopts high-grade anti-corrosion leather that meets environmental protection requirements. Due to the inherent characteristics of leather materials there will be a normal slight noise when rubbing at the relative movement, which is a normal phenomenon. Note: If you hear fabric or leather tearing sound when the machine is running, please stop using it immediately and seek professional maintenance.

Fault	The massage wheel did not reach the shoulders or neck.
Solution	If the head does not touch the head cushion or the back does not touch the backrest, the shoulder position may be lower than the actual position during the body shape detection process. Sit in the deepest position of the seat, rest your head on the headrest, and start over from the beginning.

Fault	The unit is damaged. The power cord or power plug is abnormally hot.
Solution	To prevent accidents, be sure to contact an authorized service center.

Please feel free to contact the designated service personnel for any questions or doubts about this product.

Product specifications

Model: A505-2

Name: Massage Chair

Rated voltage: 110-120V~ 60Hz

220-240V~ 50Hz/60Hz

Rated power: 100W

Rated time: 20min

Safety structure: Class I